


## MILLING ASC TI BLANKS IN HYPERDENT®

To ensure proper milling of abutments from Preat ASC Ti Blanks, please be sure to make the following adjustments in hyperDENT.

### Manually place the connector

Place the connector on the opposite side of the ti blank window

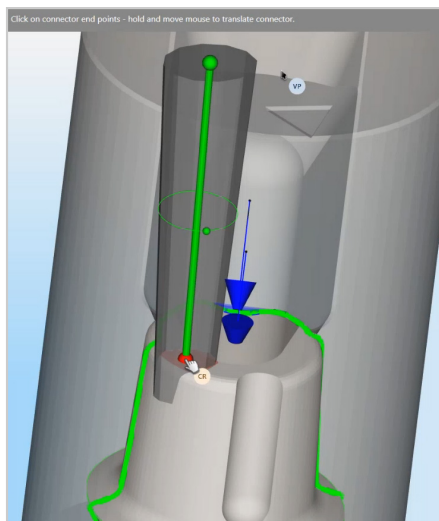

### Adjust settings

Utilize settings similar to those shown here. You may decrease the size of the connector or increase the **Milling cut** if doing so optimizes your post-processing.

**Shape type:** Ellipse

**Angle:** 6 deg

**Height:** 1.000mm

**Width:** 1.500mm

**Milling cut:** 0 %

**Cut safety distance:** 0.000mm

